

# Bond Aligner®

**Clear Aligner Adhesive for thermoplastic aligners**

**Ref# BA**

**INDICATIONS FOR USE:** Bond Aligner® is intended for use as an orthodontic bonding adhesive for brackets and appliances to thermoplastic aligner surfaces. Can also be used for forming button attachments. Bond Aligner is formulated to work with clear, thermoforming aligner plastics. It is not recommended for use with cloudy (Class C) thermoforming materials.

**INSTRUCTIONS FOR USE:**

1. Apply Bond Aligner® clear aligner adhesive to the appliance to be bonded. Aligner should be clean and free of any oral fluids or biofilm where the attachment will be bonded.
2. Seat appliance on thermoplastic aligner and light cure through the aligner for 10-20 seconds (depending on the intensity of curing light).



**PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse spill area immediately with copious amounts of water for several minutes.

Use of UV, halogen and LED curing lights can be associated with eye damage. The patient, doctor and the assistant should wear properly designed protective eyewear when using these light curing devices.



**TEMPERATURE LIMITATION:** Store at room temperature (15-30°F)

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.

**WARRANTY:** Reliance Orthodontic Products Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.



Made in the USA



*Reliance Orthodontic Products, Inc.*

Scan for more information

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 • [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)

Australian Sponsor: Emergo  
Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia

Check out our new learning  
platform Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)

EC

REP

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands