



## KOBAYASHI TIE HOOKS

V7 7/18

**INDICATIONS FOR USE:** Reliance Kobayashi Tie Hooks are intended for use in ligation of archwires and hooks to orthodontic brackets and elastics.

**PRODUCT FEATURES:**

- High grade stainless steel.
- Durable weld to form strong hook.



Single Use Device

**INSTRUCTION FOR USE**

1. Remove single Kobayashi Tie Hook from container. Perform hot or cold sterilization.
2. Position open end of the tie hook over the bracket tie wing.
3. Twist the two ends (approximately 4 times) until it is firmly seated against the bracket base.
4. Cut excess wire off close to the base without compromising the twist.
5. Bend the loop to the desired angulation.
6. Attach elastic, if desired.

Made in the USA

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**Rx Only:** U .S. Federal law restricts this device to sale by or on the order of a dental professional.



**NICKEL WARNING:** This product contains a chemical known to the State of California to cause cancer.



Flammable as dust

Check out our new learning platform  
Reliance U!  
[www.RelianceU.tech](http://www.RelianceU.tech)



*Reliance Orthodontic Products, Inc.*

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)

Australian Sponsor: Emergo Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia

Scan for more information



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands