

## Reliance POWER SLOT

**INDICATIONS FOR USE:** Reliance Power Slot is intended for use as a fast curing fiber optic Light Guide used with a compatible dental curing light.

### **PRODUCT FEATURES**

The Reliance Power Slot is designed to significantly increase the light output of the dental curing light and reduce curing time when used with light cure bonding materials.

### **INSTRUCTIONS FOR USE**

1. Firmly grasp the curing light and align the metal end with the opening on the curing curing light. A firm, positive click will be heard when seating correctly.
2. For Metal Brackets that have been positioned, place the Power Slot light guide along the flat, long edge against the tooth at a slight angle and close to the bracket base. Light cure the bracket per the adhesive manufacturer instructions for cure time.
3. For Composite or Ceramic Brackets, Seat and position the bracket. Place the power slot light guide on the facial of the bracket and cure directly through the arch wire slots per the adhesive manufacturer instructions for cure time.
4. **For Band composites: Place the Power Slot over the occlusal edge of the band and cure per the adhesive manufacturer instructions for cure time.**
5. **For Direct bond lingual retainers: Place the Power Slot over the composite and cure per the adhesive manufacturer instructions for cure time.**

**STERILIZATION:** The Power Slot can be sterilized chemically or using an autoclave. Do not use high heat sterilization, as it may damage the light director.

**TO TEST ON A RADIOMETER:** Place the Power Slot tip in the center of the radiometer's sensor. Turn the curing light on and take a reading for 3 seconds. The resulting number should be multiplied by 3 to obtain the output.

**TO REORDER:** LED PowerSlot: CAT# LEDPS

11mm Powerslot: CAT 11MMPS

Demetron Compatible Power Slot: CAT# PSD

**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

**Rx Only:** U.S. Federal law restricts this device to sale by or on the order of a dental professional.



*Reliance Orthodontic Products, Inc.*

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)

Scan for more information



EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands

Australian Sponsor: Emergo  
Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia