



Restore™
Diamond Polishing Paste
(REF# RESPP2)

V4 6/18

INDICATIONS FOR USE: Restore™ diamond polishing paste is intended for use in restoring an aesthetic polish on porcelain, zirconia or ceramic surfaces following orthodontic appliance removal.

INSTRUCTIONS FOR USE:

- 1.) Rinse the zirconia, ceramic or porcelain surface with water.
- 2.) Apply Restore™ liberally to the surface.
- 3.) Polish with a Prophyl cup or soft buff.
- 4.) Rinse with water.

Repeat process above to enhance luster.



TEMPERATURE LIMITATION: Store at room temperature (15-30°C)

CAUTION: DO NOT REUSE IF SYRINGE BECOMES CONTAMINATED. AVOID CONTACT OF THE REUSABLE PARTS WITH THE PATIENTS MOUTH. USE NEW, UNCONTAMINATED GLOVES. APPLY DISPOSABLE BARRIER SLEEVES/WRAPS BEFORE EACH PATIENT.



Avoid contact with skin and eyes.

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontics Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from use of or suitability of the product for its intended use. The user assumes all risk and liability in connections therewith.

Rx Only: U.S. Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.


1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 • www.RelianceOrthodontics.com



Scan for more information

Australian Sponsor: Emergo Australia
Level 20, Tower II, Darling Park
201 Sussex St.
Sydney, NSW 2000 Australia

Made in the USA

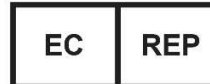

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Check out our new learning
platform Reliance U!
www.RelianceU.tech